

**WAR OFFICE MEDICAL DIVISION  
FIRST FIELD DRESSING.**

Outer cover-Pull tapes apart.

**TO OPEN**

Inner waterproof cover-Tear apart  
Uncemented corners as indicated by  
the arrow.

**CONTENTS**-Two Dressings In  
waterproof covers, each consisting  
of a Gauze Pad, stitched to a  
Bandage, and a Safety Pin.

**DIRECTIONS FOR USE**-Take the  
Folded ends of  
the Bandage in each hand, and  
keeping the  
Bandage taut, apply the Gauze Pad  
to the  
wound and fix the bandage. One  
Dressing  
to be used for each wound.  
**DO NOT HANDLE THE GAUZE OR  
WOUND.**

**CUXSON, GERRARD & CO. LTD,  
JUNE 1915. OLDBURY,  
BIRMINGHAM.**

**DIVISION MÉDICALE DU MINISTÈRE  
DE LA GUERRE  
TROUSSE DE PREMIERS SOINS DE  
CAMPAGNE**

Enveloppe extérieure – Tirez sur les  
rubans.

**POUR OUVRIR**

Enveloppe intérieur hydrofuge. -  
Déchirez.  
Détachez les coins tel qu'illustré par  
la flèche.

**CONTENU** – Deux troussees sous  
enveloppe hydrofuge, contenant  
chacune un tampon de gaze, cousu  
sur un pansement, et une épingle.

**MODE D'EMPLOI** – Agrippez les  
bouts repliés du pansement, tenez-  
le bien tendu et appliquez le gaze  
sur la blessure, puis fixez le  
pansement en place. Utilisez une  
trousse pour chacune des  
blessures.

**NE TOUCHEZ NI AU GAZE, NI À LA  
BLESSURE.**

**CUXSON, GERRARD & CO. LTD,  
JUN 1915. OLDBURY,  
BIRMINGHAM.**